



AccuDiag™ Crypto/Giardia Ag Combo (Fecal) ELISA Kit

REF 8310-3

IVD See External Label 2-8°C 96 Tests

Crypto/Giardia Combo ELISA	
Principle	Sandwich ELISA
Detection	Qualitative
Sample	Stool Sample
Incubation Time	100 minutes
Sensitivity	99%
Specificity	100%
Shelf Life	12 Months from the manufacturing date

*Laboratory results can never be the only basis of a medical report. The patient history and further tests have to be taken into account.

PRODUCT FEATURES

- Very easy to use with little training
- Highly specific and consistent Assay
- Provides accurate results quickly
- Reading of results both visually and as absorbance data

INTENDED USE

This ELISA is an *in vitro* immunoassay for the qualitative determination of *Giardia* and *Cryptosporidium* antigen in feces.

SIGNIFICANCE AND SUMMARY

Giardia lamblia is the protozoan parasite responsible for the disease giardiasis. Symptoms of acute giardiasis include diarrhea, nausea, weight loss, malabsorption, abdominal cramps, flatulence and anemia.¹ The disease may manifest itself as an acute, chronic or as an asymptomatic infection. Giardiasis is the most prevalent parasitic disease in the United States and is responsible for

an estimated 100 million mild infections and 1 million severe infections each year.¹⁶

The mode of transmission of *Giardia* is through fecal-oral ingestion of cysts. Epidemics of giardiasis have been documented in day care centers and by drinking contaminated water.¹² Day care centers may be directly or indirectly responsible for 45% of diagnosed *Giardia* infections in the United States.⁴ One study found 54% of the children at a day care center were infected.¹

Cryptosporidium is a coccidian parasite that is recognized as an important enteric pathogen. The organism causes an acute, though self-limiting infection in immunocompetent individuals. Incubation periods of 1 to 12 days have been reported with most oocyst shedding ending by day 21. Symptoms range from mild to severe diarrhea with a variety of complications.^{8,15,21,22,23,24}

The infection in immunocompromised patients is much more severe and may often be life threatening. Passage of fluid, up to 12 liters per day, has been reported.^{8,9,10,25,26,27}

Multiple pathways of *Cryptosporidium* transmission have been implicated. These include animal to human, water contamination and person-to-person. The latter may include contact between members of the same household and day care centers.^{8,9,25,26,27}

Diagnosis of *Giardia* and *Cryptosporidium* infections has been done through a number of invasive and non-invasive techniques. Of the non-invasive techniques, microscopic examination of stools has been the most common. However, this method relies on an experienced technician and subsequent observation of intact organisms. Because of the historically low proficiency of correct microscopic examinations and intermittent excretion of organisms, alternative diagnostic methods have been investigated.^{3,5,6,11,12,17,18,27,28}

One important alternative has been the development of an antigen capture enzyme linked immunosorbent assay (ELISA) for use with stools. These tests have shown comparable sensitivity to experienced microscopic examinations, are fairly simple to perform and do not require the observation of intact organisms.^{5,6,13,14,19,20}

ASSAY PRINCIPLE

During the first incubation, *Giardia* and/or *Cryptosporidium* antigens present in stool specimens are captured by antibodies attached to the microwells. The wells are incubated and washed before anti-*Giardia* and anti-*Cryptosporidium* antibodies conjugated to peroxidase are added. The enzyme conjugate will "sandwich" any antigen bound to the wells. After washings to remove any unbound enzymes, a Chromogen is added, which develops a blue color in the presence of the enzyme complex. The Stop Solution ends the reaction and turns the blue color to yellow. If no antigen is captured, or if there is an insufficient level of antigen, no colored reaction will take place.

SPECIMEN COLLECTION AND PREPARATION

- No modification of collection techniques used for standard microscopic O&P examinations is needed.
- Stool samples may be used as unpreserved or frozen, in Cary-Blair Transport Medium or in preservation media of 10% formalin or SAF.
- Unpreserved samples should be kept at 2-8°C and tested within 24 hours of collection. Samples that cannot be tested within this time should be frozen at -20°C or lower until used. Avoid multiple freeze/thaw cycles.



- Formalinized and SAF preserved samples may be kept at room temperature (15-25 °C) or at 2-8 °C and tested within 18 months of collection. DO NOT freeze preserved samples.
- Samples in Cary-Blair should be kept at 2-8 °C or -20 °C and tested within 1 week of collection. Avoid multiple freeze/thaw cycles.

REAGENTS

Materials provided with the kit

- Giardia/Cryptosporidium* Stool Antigen Microwell ELISA Kit
- Test Strips: Microwells containing anti-*Giardia* and anti-*Cryptosporidium* antibodies - 96 test wells in a strip holder.
- Enzyme Conjugate: One (1) bottle containing 11 ml of anti-*Giardia* and anti-*Cryptosporidium* antibodies conjugated to horseradish peroxidase with Thimerosal.
- Giardia* Positive Control: One (1) vial containing 2 ml of a diluted *Giardia* positive formalinized stool supernatant.
- Cryptosporidium* Positive Control: One (1) vial containing 2 ml of a diluted *Cryptosporidium* positive formalinized stool supernatant.
- Negative Control: One (1) vial containing 2 ml of buffered protein solution with Thimerosal.
- Chromogen: One (1) bottle containing 11 ml of the chromogen tetramethylbenzidine (TMB) and peroxide.
- Dilution Buffer: Four (4) bottles containing 30 ml of a buffered protein solution with Thimerosal.
- Wash Concentrate (20X): Two (2) bottles containing 25 ml of concentrated buffer with surfactant and Thimerosal.
- Stop Solution: One (1) bottle containing 11 ml of 5% phosphoric acid.

Materials required but not provided

- Transfer Pipettes
- Squeeze bottle for washing strips (narrow tip is recommended)
- Graduated Cylinder
- Reagent grade (DI) water
- Micropipette
- Applicator sticks (recommended) or swabs for sample preparation
- Sample dilution tubes

Suggested Equipment

- ELISA plate reader capable of reading dichromatically at 450/620-650 nm.

REAGENT PREPARATION

- All incubations are at room temperature (15 to 25°C)
- Ensure all samples and reagents are at room temperature (15-25°C) before use. Frozen samples must be thawed completely before use.
- All dilutions of stools must be made with the Dilution Buffer provided. Do not use dilution buffer from a kit with a different lot number.
- If needed, prepared samples can be centrifuged at 2000-3000 g for 5-10 minutes. Ensure supernatant is clear before use.
- When running the assay, try to avoid the formation of bubbles in the wells. Bubbles may affect overall performance and reading of end results. Slapping the wells out on a clean absorbent towel after each wash step should help to minimize bubbles in the wells.
- Controls must be diluted each time the kit is run. Controls are provided ready to use. DO NOT dilute further.
- Unpreserved and Preserved specimens have different testing procedures. See below for specific instructions on how to run the assay using each procedure.

ASSAY PROCEDURE

Preserved specimen procedure:

- For samples in SAF, 10% Formalin or Cary-Blair, mix contents thoroughly inside container. No further processing is required.
- Break off the required number of wells needed (number of samples plus 2 for controls) and place in holder.
- Using a micropipette, add 100 µl of negative control to well # 1 and 100 µl of positive control to well # 2.
- Using a micropipette, add 50 µl of Dilution Buffer to each sample well. DO NOT add Dilution Buffer to control wells.
- Add 50 µl of sample to each sample well with Dilution Buffer.
- Incubate for 60 minutes at room temperature (15-25°C), then wash.* After last wash, slap the wells out on a clean absorbent towel to remove excess wash buffer.
- Add 2 drops of Enzyme Conjugate to each well.
- Incubate for 30 minutes at room temperature (15-25°C), then wash.* After last wash, slap the wells out on a clean absorbent towel to remove excess wash buffer.
- Add 2 drops of Chromogen to each well.
- Incubate for 10 minutes at room temperature (15-25°C).
- Add 2 drops of Stop Solution to each well. Mix wells by gently tapping the side of the strip holder with index finger for approximately 15 seconds. Read reaction within 5 minutes after adding stop solution.
- Read results visually or using an ELISA plate reader (see instructions below).

Unpreserved specimen procedure:

- Prepare sample dilutions in tubes using 0.7 ml of Dilution Buffer and 0.1 g, about the size of a small pea, of fecal sample using an applicator stick. Mix thoroughly before using. **-IF USING SWABS**, add 1 ml of dilution buffer to dilution tube. Coat the swab with a thin layer of specimen and mix into dilution buffer, expressing as much fluid as possible. Mix thoroughly before using
- For watery unpreserved specimens, mix contents then add 0.1 ml of sample to 0.7 ml of Dilution Buffer in dilution tubes. Mix thoroughly before using.
- Break off the required number of wells needed (number of samples plus 2 for controls) and place in holder.
- Using a micropipette, add 100 µl of negative control to well # 1.
- Using a micropipette, add 100 µl of positive control to well # 2.
- Add 100 µl of diluted sample to each well.
- Incubate for 60 minutes at room temperature (15-25°C), then wash.* After last wash, slap the wells out on a clean absorbent towel to remove excess wash buffer.
- Add 2 drops of Enzyme Conjugate to each well.
- Incubate for 30 minutes at room temperature (15-25°C), then wash.* After last wash, slap the wells out on a clean absorbent towel to remove excess wash buffer.
- Add 2 drops of Chromogen to each well.
- Incubate for 10 minutes at room temperature (15-25°C).
- Add 2 drops of Stop Solution to each well. Mix wells by gently tapping the side of the strip holder with index finger for approximately 15 seconds. Read reaction within 5 minutes after adding stop solution.
- Read results visually or using an ELISA plate reader (see instructions below).

**Washings consist of vigorously filling each well to overflowing and decanting contents five (5) separate times. When possible, avoid formation of bubbles in the wells as this may affect the end results.*



TROUBLESHOOTING

- **Problem:** Negative control has excessive color after development.
- **Reason:** Inadequate washings.
- **Correction:** Wash more vigorously. Remove excessive liquid from the wells by tapping against an Absorbent towel. Do not allow test wells to dry out.

RESULTS AND INTERPRETATION

Interpretation of Results - Visual

- **Positive:** Any sample well that is obviously more yellow than the negative control well.
- **Negative:** Any sample well that is not obviously more yellow than the negative control well.

NOTE: The negative control, as well as some samples, may show some slight color. A sample well must be obviously darker than the negative control well to be called a positive result.

Interpretation of Results - ELISA Reader

- Zero reader on air. Read all wells using a bichromatic reading with filters at 450 nm and 620-650 nm.
- **Positive:** Absorbance reading of 0.08 OD units and above indicates the sample contains *Giardia* and/or *Cryptosporidium* antigen.
- **Negative:** Absorbance reading less than 0.08 OD units indicates the sample does not contain detectable levels of *Giardia* or *Cryptosporidium* antigen.

QUALITY CONTROL

The negative control and positive controls must be included each time the assay is run. The use of positive and negative controls allows easy validation of kit stability readings at dual wavelength of 450/620-650.

- Negative - 0.0 to 0.8 OD units
- Positive - 0.5 OD units and above

EXPECTED RANGES OF VALUE

Normal healthy individuals should be free of *Giardia* and *Cryptosporidium* and should test negative. A positive reaction indicates that the patient is shedding detectable amounts of either antigen.

PERFORMANCE CHARACTERISTICS

Study #1

A study was performed with the Diagnostic Automation, Inc. G/C Combo assay using 135 fresh/frozen specimens confirmed positive or negative by microscopy. Of the 135 specimens 12 were confirmed positive for either *Cryptosporidium* or *Giardia*, and 123 specimens were confirmed negative for both *Cryptosporidium* and *Giardia*. The results from the study are shown in the table below.

		Microscopy	
		+	-
DAI	+	12	3
	-	0	120

Sensitivity: 100% (12/12)
Specificity: 97.6% (120/123)

Study #2

		Giardia	
		DAI	
		+	-
Reference Method*	+	35	0
	-	0	95

Positive Agreement: 100% (35/35)
Negative Agreement: 100% (95/95)

		Cryptosporidium	
		DAI	
		+	-
Reference Method*	+	52	0
	-	1	95

Positive Agreement: 100% (52/52)
Negative Agreement: 99% (95/96)

		Cryptosporidium/Giardia	
		DAI	
		+	-
Reference Method*	+	11	0
	-	1	95

Positive Agreement: 100% (11/11)
Negative Agreement: 99% (95/96)

*Reference Method refers to a commercially available ELISA.

LIMITATIONS OF THE ASSAY

Test results should be used as an aid in diagnosis and should not be interpreted as diagnostic by themselves. DO NOT concentrate stool samples. Assay will not give accurate results on a concentrated sample. A negative result can occur from an antigen level lower than the detection limits of this assay. Multiple samples over time may be indicated for those patients that are suspected of being positive for *Giardia* or *Cryptosporidium*.

PRECAUTIONS

- Do not deviate from the specified procedures when performing this assay. All specimen dilutions, incubation times/temperatures and washings have been optimized for the best performance characteristics. Deviations from the specified procedures may affect the sensitivity and specificity of the assay.
- For In Vitro Diagnostic Use Only.
- Do not interchange reagents between kits with different lot numbers.
- Do not use reagents that are beyond their expiration dates. Expiration dates are on each reagent label. Use of reagents beyond their expiration dates may affect results.
- Unused microwells should be stored in the desiccated pouch to protect them from moisture.



- Do not use solutions if they precipitate or become cloudy. **Exception:** Wash concentrate may precipitate during refrigerated storage, but will dissolve upon warming.
- Do not add azides to the samples or any of the reagents.
- Controls and some reagents contain thimerosal as a preservative, which may be irritating to skin, eyes and mucous membranes. In case of contact, flush eyes or rinse skin with copious amounts of water.
- Treat all reagents and samples as potentially infectious materials. Use care to prevent aerosols and decontaminate any spills of samples.
- Stop solution is a 5% solution of phosphoric acid in water. If spilled on the skin, wash with copious amounts of water. If acid gets into the eyes, wash with copious amounts of water and seek medical attention.
- Persons who are color blind or visually impaired may not be able to read the test visually and should use spectrophotometric readings to interpret results.

STORAGE CONDITIONS

- Reagents, strips and bottled components should be stored at 2-8°C.
- Squeeze bottle containing diluted wash buffer may be stored at room temperature (15-25°C).

REPRODUCIBILITY

- The intra-assay (well to well) CV was calculated using 3 *Cryptosporidium* positive, 3 *Giardia* positive and 4 *Giardia/Cryptosporidium* negative samples assayed 10 times in a single run. The mean CV for all samples was 11.8%.
- The inter-assay (run to run) CV was calculated using 3 *Cryptosporidium* positive, 3 *Giardia* positive and 4 *Giardia/Cryptosporidium* negative samples assayed on three separate days. The mean CV for all samples was 7.5%.

CROSS REACTIVITY

No cross-reactions were seen with the following organisms:

Entamoeba hartmanni, *Endolimax nana*, *Entamoeba histolytica/dispar*, *Entamoeba coli*, *Blastocystis hominis*, *Dientamoeba fragilis*, *Chilomastix mesnili*, *Strongyloides stercoralis*, *Ascaris lumbricoides*, *Enterobius vermicularis*, *Diphyllobothrium* species, *Hymenolepis nana*, *Clonorchis sinensis*, *Enteromonas hominis*, *Trichuris trichiura*, *Iodamoeba buetschlii*, Hookworm, *Schistosoma mansoni*, rotavirus, *Taenia* eggs, *Fasciola* eggs, *Isospora belli*, *Entamoeba polecki*, adenovirus, & 33 bacterial species (list available on request).

REFERENCES

- Black, R. et al. "Giardiasis in Day-Care Center: Evidence of Person to Person Transmission." *Pediatric*, #60 (4), October 1977, pp. 486-491.
- Craun, G. "Waterborne Giardiasis in the United States." *Lancet*, August 30, 1986, pp. 513-514.
- Smith, J. "Identification of Fecal Parasites in the Special Parasitology Survey of the College of American Pathologist." *ALCP*, Vol. 72 (2) August 1979, pp. 371-373.
- Kappus, K. and Juranek, D. "Giardia in the Well." *JAMA*, March 25, 1988, Vol. 259 (12), pp. 1810.
- Allison, M.C. et al. "A Microscopic and Immunodiagnostic Search for Giardiasis in Patients with Gastrointestinal Disorders." *Scan J Gastroenterol*, 1988 #23, pp. 209-212.
- Nash, T., Herrington, D. and Levine, M. "Usefulness of and Enzyme Linked Immunosorbent Assay for Detection of *Giardia* Antigen in Feces." *J Clin Micro*, July 1987, Vol. 25 (7), pp. 1169-1171.
- Peters, C. et al. "Prevalence of Enteric Parasites in Homosexual Patients Attending An Outpatient Clinic." *J Clin Micro*, Oct. 1986, Vol. 24 (4), pp. 684-685.
- Chapman, P.A. "Cryptosporidiosis: Recent Trends in Epidemiology, Diagnosis, and Treatment." *Serodiag & Immunother Infect Dis #2*, 1988, pp. 311-317.
- Meyer, E.A. "Waterborne *Giardia* and *Cryptosporidium*." *Parasit Today*. Vol. 4, #7, 1988, pp. 200-201.
- Garcia, L., Bruckner, D., Brewer, T., "Cryptosporidiosis in Patients with AIDS." *ACPR*, May 1988, pp. 38-41.
- Stibbs, H., Ongerth, J. "Immunofluorescence Detection of *Cryptosporidium* Oocysts in Fecal Smears." *J Clin Micro*, Vol 24 #4, Oct. 1986, pp. 517-521.
- McLaughlin, J. et al. "Identification of *Cryptosporidium* Oocysts by Monoclonal Antibody." *Lancet*, January 3, 1987, pp. 51.
- Ungar, B. "Enzyme-Linked Immunoassay for Detection of *Cryptosporidium* Antigens in Fecal Specimens." *J Clin Micro*, Vol. 28 #11, Nov 1990, pp. 2491-2495.
- Anusz, K., et al. "Detection of *Cryptosporidium parvum* Oocysts in Bovine Feces by Monoclonal Antibody Capture Enzyme-Linked Immunosorbent Assay." *J. Clin Micro*, Vol. 28 #12, Dec. 1990, pp. 2770-2774.
- Jokipii, L., et al. "Cryptosporidium: A Frequent Finding In Patients With Gastrointestinal Symptoms." *Lancet*, August 13, 1985, pp. 358-360.
- Intestinal Protozoa "Cinderellas of Parasitology." *ASM News*, Vol. 52 (10), 1986, pp. 521-522.
- Burke, J. "Giardiasis in Childhood." *Am J Dis Child*, Nov. 1975, Vol. 129, pp. 1304-1310.
- Danciger, M. and Lopez, M. "Numbers of *Giardia* in the Feces of Infected Children." *Am J Trop Med Hyg.*, Vol. 24 (2), 1975, pp. 237-242.
- Green, E. Miles, M., and Warhurst, D. "Immunodiagnostic Detection of *Giardia* Antigen in Feces by a Rapid Visual Enzyme Linked Immunosorbent Assay." *Lancet*, Sept. 28, 1985, pp. 691-693.
- Ungar, B et al. "Enzyme Linked Immunosorbent Assay for the Detection of *Giardia Lamblia* in Fecal Specimens." *J Infect Dis.*, January 1984, Vol. 149 (1), pp. 90-97.
- Shephard, R., et al. "Shedding of Oocysts of *Cryptosporidium* in Immunocompetent Patients." *J Clin Pathol*, Vol. 41, 1988, pp. 1104-1106.
- Holten-Anderson, W., et al. "Prevalence of *Cryptosporidium* Among Patients with Acute Enteric Infection." *J. Infect*, Vol. 9, 1984, pp. 277-282.
- Jokipii, L. and Jokipii, M. "Timing of Symptoms and Oocyst Excretion on Human Cryptosporidiosis." *N Engl J Med*, Vol. 315 #26, 1986, pp. 1643-1647.
- Hart, M. et al. "Acute Self-Limited Colitis Associated with *Cryptosporidium* in an Immunocompetent Patient." *J Ped Gastro Nutr*, Vol. 8, 1989, pp. 401-403.
- Egger, M. et al. "Symptoms and Transmission of Intestinal Cryptosporidiosis." *Arch Dis Child*, Vol. 65, pp. 445-447.
- Nwanyanwu, O., et al. "Cryptosporidiosis in a Day-Care Center." *Texas Med*, Vol. 85, June 1989, pp. 40-43.
- Current, W. and Garcia, L. "Cryptosporidiosis." *Clin Micro Rev*, Vol. 4 #3, July 1991, pp. 325-358.
- Weber, R. et al. "Threshold of Detection of *Cryptosporidium* Oocysts in Human Stool Specimens; Evidence for Low Sensitivity of Current Diagnostic Methods." *J Clin Micro*, Vol. 29 #7, July 1991, pp. 1323-1327.



MANUFACTURER AND BRAND DETAILS

ISO 13485:2016



ISO 13485
Quality
Management for
Medical Devices
CERTIFIED

Diagnostic Automation/Cortez Diagnostics, Inc.
21250 Califa Street, Suite 102 and 116,
Woodland Hills, California 91367 USA

Date Adopted	2023-07
Brand Name	AccuDiag™
REF 8310-3	AccuDiag™ - Crypto/Giardia Ag Combo ELISA
EC REP	CEpartner4U, Esdoornlaan 13, 3951 DB Maarn, The Netherlands. www.cepartner4u.eu
Revision Date: 2020-08	